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GB English version

What is this program for?

****LIVE SLIT LAMP PIC**** is designed to work with ****ophthalmic slit lamps**** that can save photos to a chosen folder – locally or over the network.

The program automatically detects new files in watched folders and displays them on screen. This allows the doctor to ****instantly view patient photos**** without manual refreshing.

How does it work?

* In ****START**** mode the program checks every few seconds if a new photo has appeared in the folder.

* The newest photo is automatically shown in the ****right preview window****.

* The ****left panel**** shows thumbnails of the latest photos for manual browsing.

How to start?

1. Open the program folder and click the icon to run it.
2. (Optional) place the ****eye.ico**** file next to the program – the window will display this icon.
3. In ****Settings****, add the folder (local or network) where the slit lamp saves its images.

What do you see in the window?

* ****Left panel**** – photo thumbnails (newest on top). Click to view.

* ****Right panel**** – large preview of the selected photo.

* ****Top bar**** – number of photos and the `Today only` toggle.

* ****Bottom bar**** with buttons:

* ****START**** – enable automatic checking and loading of new photos,

* ****STOP**** – disable automatic checking,

* ****Settings**** – program settings (folders, parameters),

* ****Open latest**** – instantly show the newest photo,

* ****Quit**** – close the program.

Step by step usage

* ****Click a thumbnail**** on the left – the large preview appears on the right.

* ****Double-click**** – opens the photo in the system's default viewer.

* **Mouse wheel** – zoom in and out.

* **Drag the photo** (left mouse button) – move the zoomed image.

* **Keyboard shortcuts**:

* ↑/↓ – switch between photos,

* PgUp/PgDn – scroll the list of thumbnails,

* Home/End – jump to oldest/newest photo,

* Space – toggle automatic refresh on/off.

Settings

In the **Settings** menu you can:

* select folders where photos are saved (e.g. the slit lamp's network folder),

* define how many of the latest photos should be visible (e.g. 20),

* set the auto-refresh interval (e.g. every 5 seconds),

* choose the quality of thumbnails and preview images.